



Best Ears

In-Ear Monitors

Audiologist's Info Sheet

Hello and thank you for taking good care of our customers! We will be using their ear impression to make Custom In-The-Ear Monitors (Earpieces) otherwise known as "IEMs". In order for us to make them the best that they can be, we have a couple of requests from you while you are taking their impression:

- **Best Ears** IEMs. require a full impression and must include the crus of the helix, antitragus and tragus (just passing the second bend of the ear canal)
- Foam or cotton ear dams are acceptable for use, however foam ear dams are preferred
- Ear impressions should taken "open mouth" and using a bite block is preferred. If not, chewing or just holding the mouth open is also acceptable. It is also recommended that the customer make any faces that he/she may make when performing to ensure a tight fit no matter what.
- Please use as much molding material as you're willing. It's great if we have way too much to work with rather than the impressions being to shallow and us needing to have them redone.
- Should you have any questions, please don't hesitate to call me personally (Michal) on my mobile, which is: 937.232.5792

Thank you for your efforts to make another quality pair of **Best Ears** In-Ear Monitors